

# Predicta® desensitizer

## BIOACTIVE



## Fast and Effective Relief from Dentin Hypersensitivity

Predicta® Bioactive Desensitizer uses the elements that are naturally found in the tooth, (**calcium** and **phosphate**) and restores them to the tooth in the form of hydroxyapatite plugs that immediately start forming in the dentinal tubules upon application.


Predicta Bioactive Desensitizer doesn't stop there. An additional layer of mineralization also forms atop the dentinal tubules as a result of the ongoing release of calcium and phosphate ions. This extra layer broadens the seal that protects the pain-sensing A-delta nerve fibers in the pulp from being stimulated.

### Treats Dentin Hypersensitivity Due to:


- ▶ Exposed roots after perio-surgery
- ▶ Bleaching procedures
- ▶ Scaling or root planing
- ▶ Gingival recession

### Bioactive, biocompatible.

This syringe dispensed, ready-to-use gel is considered highly biocompatible as it contains no aldehydes, methacrylates or other soft-tissue irritating chemicals.

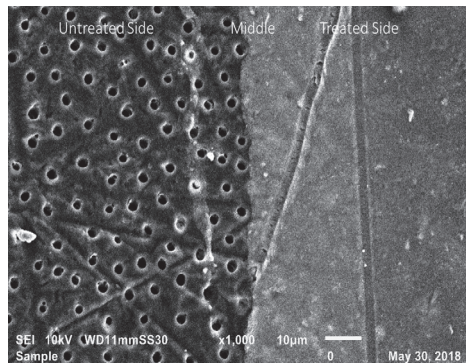

Comparison of untreated dentinal tubules (left), with tubules that have been covered with a dense layer of Predicta® Bioactive Desensitizer (right). Images courtesy of University of Washington School of Dentistry.

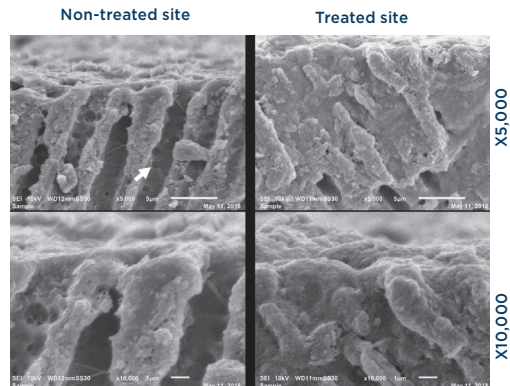

Cross-sectional SEM images. Note the patency of dentinal tubules and open lateral branch orifices (arrow) that have been well sealed on the treated side. (Image courtesy of Dr. Alireza Sadr, DDS, PhD, University of Washington School of Dentistry)

“

The ease with which I can treat my patients' sensitive teeth by using Predicta desensitiser has positively surprised me. The gel is incredibly easy to use and very gentle on the patient. It gives immediate relief from hypersensitivity that lasts for a long time. The dental hygienists in our practice also enjoy using it on their patients.

”



Sjobbe Besseling, DDS, Netherlands

Scan the QR code to watch the video



“Predicta Desensitizer is very effective, it works very well immediately blocking the sensitivity for several days and with a second application it dramatically reduces sensitivity in the long term.”

Dr. Consuelo Sanavia, Dental Hygienist, Italy



### ITEM Predicta® Bioactive Desensitizer Kits

S626	<b>Single Syringe Kit</b> (1) 3 ml syringe, (10) applicator tips, (15) applicator brushes, (1) mixing well
S625	<b>Three Syringe Kit</b> (3) 3 ml syringes, (30) applicator tips, (50) applicator brushes, (1) mixing well

### Accessories & Refills

S390	<b>Dispensing Tip (Pre-Bent/Yellow) Refill</b> (30 pcs)
S391	<b>Fine (1.5mm) Micro Brush Applicator Refill</b> (100 pcs)



Scan the QR code to learn more